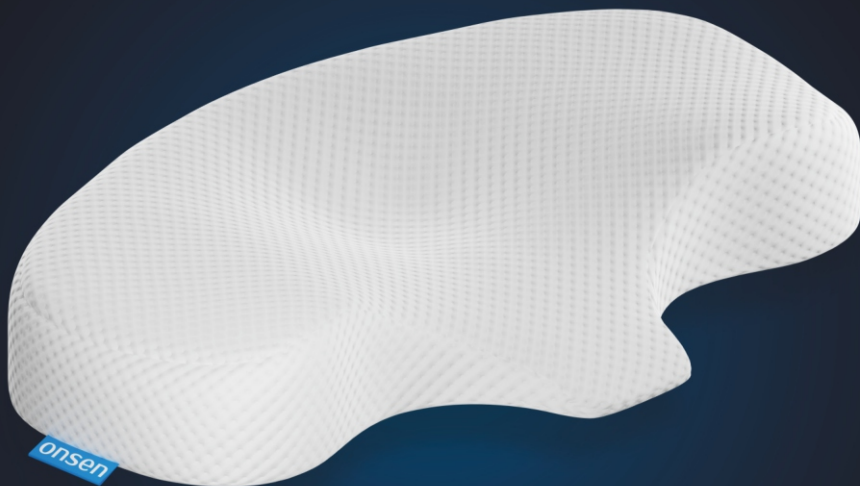
onsen

**10**  
**YEAR**  
WARRANTY

## User Guide

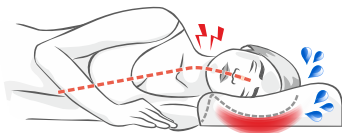
Gebrauchs-anweisung | Instrucciones del usuario  
Mode d'emploi | Istruzioni per l'uso  
Gebruikershandleiding | Instrukcja obsługi

**A NEW GENERATION**  
of open-cell foam  
by **ONSEN**

# ASANA


Poduszka ortopedyczna  
Orthopedic Pillow  
Orthopädisches Kissen

# Born to sleep

**MEMORY FOAM****SINKING OF THE HEAD**

**NO PROPER SUPPORT  
FOR THE CERVICAL SPINE**

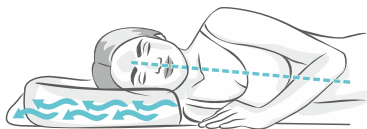**CLOSED-CELL FOAM BLOCKS AIRFLOW,  
CAUSING OVERHEATING**

**ONSEN FOAM****HEAD IS PROPERLY SUPPORTED**

**CORRECT POSITION  
OF THE CERVICAL SPINE**

**OPEN-CELL FOAM  
ALLOWS FREE AIRFLOW**

**Indications:**

- Arthritic pain of the cervical spine, discopathic changes of the spine
- Cervical hyperlordosis
- Non-specific nape and neck pain, certain types of headaches
- Degeneration of the intervertebral joints of the cervical spine

**Advantages:**

It causes no overheating and sweating of the head, which was the main disadvantage of visco-elastic foams used in pillows so far (the so-called memory foams/thermoelastic foams).

**Pillow properties:**

Owing to its specially designed shape, the pillow adapts to the curvature of the neck section of the spine, keeping it in a neutral, physiological position. This has a positive effect on the prevention of various diseases of the cervical spine. The core of the pillow is made of **ONSEN** high resilience foam with open cells. With these features it both enables excellent stabilisation and ensures free airflow, which guarantees a good night's sleep.

This is a **medical device** intended for use by non-professional users. You must use it according to this user manual. The product should be used by one patient. If you encounter any difficulties with fitting or using the product, please contact the manufacturer or an orthopedic doctor. The cushion has no contraindications for use. Its use does not require training. In case of worsening symptoms, discontinue use and immediately contact an orthopedic doctor. The product is registered in the EUDAMED database (PL-MF-000037523).

**Pillow cleaning:****Foam:**

The pillow core (foam) should be hand washed at maximum 30 °C using a mild detergent. After washing, the accumulated water should be gently squeezed out (do not wring, fold or crease the pillow) and then the pillow should be left to dry at room temperature.

**The foam should not** be washed in a washing machine nor dried in a dryer or washer-dryer.

**Pillowcase:**

The pillowcase may be washed in a washing machine at maximum 40 °C using standard washing agents.

**Storage:**

The pillow must not be folded or creased and must be stored and transported unfolded. Store it in a dry place.

**Do not** expose the pillow to direct sunlight.

**Do not** expose the pillow to high or low temperatures (above 40 °C and below 8 °C).

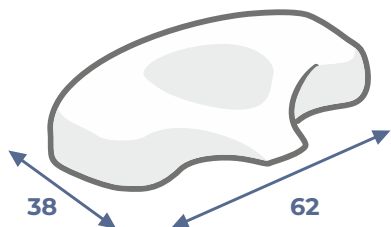
30°C

40°C


## UNIVERSAL SIZE

**Pillow dimensions:**  
width 62 cm, length 38 cm,  
max. height 10 cm,  
(tolerance +/- 2 cm).

### Sleeping positions on the ASANA pillow



#### On your back

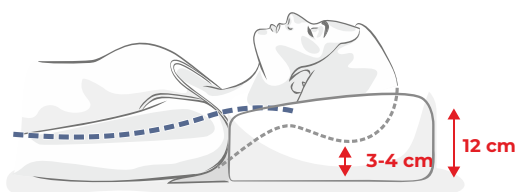
When sleeping on your back, your head should rest in the concave section in the centre of the pillow. In this position it is raised 3 to 4 cm above the mattress. This position of the head guarantees ideal alignment of the neck section of the spine. This position provides comfort and relaxation without causing pain in the neck. The specially designed "tongue" perfectly supports the thoracic section of the spine.

#### On your side

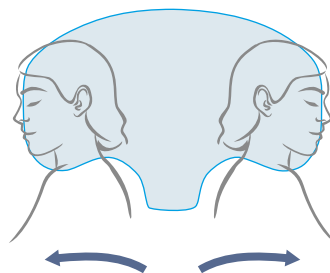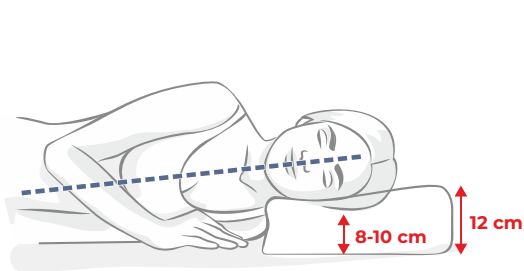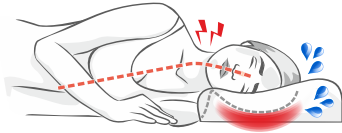Depending on your height and weight and the firmness of the mattress, the distance between your head and the mattress should be between 8 and 12 cm. **The contour of the face** (forehead, nose, mouth, and chin) should protrude slightly beyond the side edge of the pillow. This positioning of the head ensures its proper resting and prevents the head from sliding into the central recess of the pillow. It also provides ideal conditions for breathing smoothly while sleeping.

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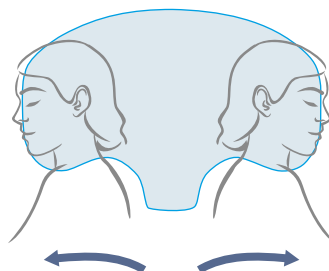
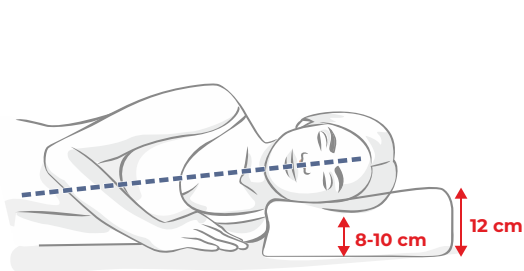
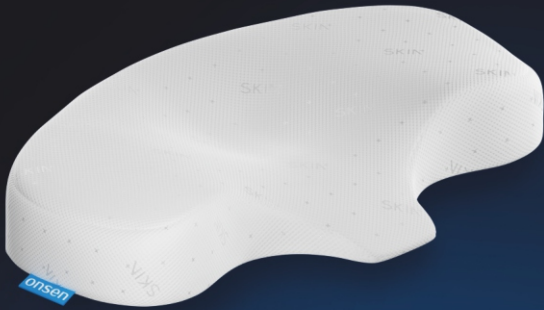
## TECHNOLOGY

### OVERNIGHT SKIN CARE

**Skin+** is a breakthrough in skin care - an innovative fabric treatment based on patent-protected probiotic technology.

**Skin+** is a textile material enhanced with carotenoid probiotics with their daily dosage comparable to the effect produced by everyday application of a carotenoid cream.

**The Skin+ fabric** contains nanocapsules with the proprietary probiotic formula of 'Bacillus indicus Pd01'. As a result of the friction between the body and the fabric, the capsules open up, the probiotics multiply and, as a consequence of the inherent feature of their metabolism, produce carotenoids, which float in the air and get attracted by our skin.

Carotenoids stimulate fibroblasts to synthesize collagen and elastin, resulting in smoother skin wrinkles.

Fabrics of the highest quality,  
ECO TEX STANDARD certified.

Foam highest quality,  
ISO 9001: 2009 certified.

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The pillow has been entirely designed and manufactured in Poland with the use of the best European materials.